


# Momentary Microphone Switch Adapter

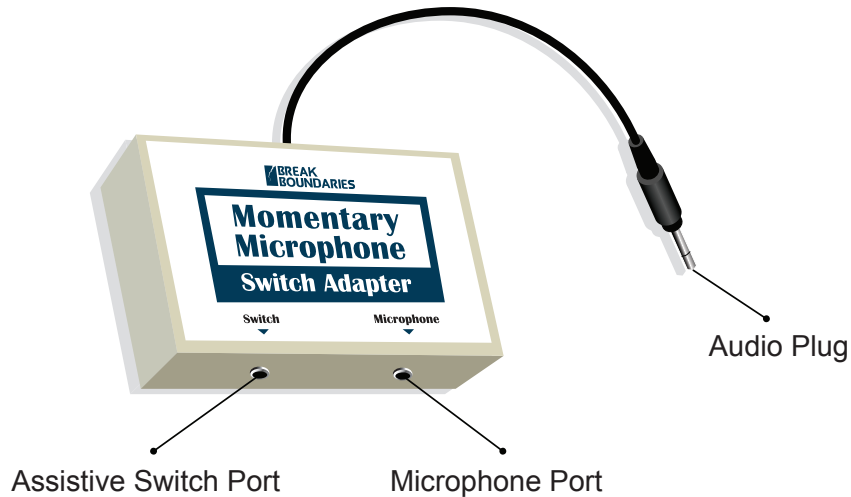

## Introduction

The Momentary Microphone Switch Adapter allows you to easily turn your microphone on and off with any assistive switch.\* Ideal for using speech recognition in noisy environments, when a tv or stereo are on, or when people are having a conversation.

\* Assistive switches come in a variety of shapes and sizes depending on the action used to activate them (such as sip-puff, pushing, pulling, pressing, blinking or squeezing). For push or press switches, the amount of force required to operate them varies or in some cases can be adjusted. Some switches may only require a touch (not even a press), some are soft and designed to be pressed with the side of one's head. A switch can be operated by almost any body part that is able to produce consistent and voluntary movement.

The Momentary Microphone Switch Adapter contains the following parts:  
Audio Plug, Assistive Switch Port, Microphone Port, Lapel Microphone.

## Setup

- Attach the Lapel Microphone to your shirt close to your mouth.
- Insert the Audio plug of the Momentary Microphone Switch Adapter into the Microphone Port on the REACH ECU.
- Insert the Audio plug of the Lapel Microphone into the Microphone Port on the Momentary Microphone Switch Adapter.
- Insert the plug from an Assistive Switch into the Assistive Switch Port on the Momentary Microphone Switch Adapter.



## Use

- Engage Assistive Switch and hold, speak voice command for REACH function, then release Assistive Switch.